

BIAKŌS™

ANTIMICROBIAL SKIN & WOUND CLEANSER



Description

BIAKŌS™ Antimicrobial Skin & Wound Cleanser helps in the mechanical removal of debris and foreign material from the skin, wound, or application site. BIAKŌS™ Antimicrobial Skin and Wound Cleanser is a pure, colorless, isotonic cleanser that is safe.

The preservative, PHMB, at a concentration of 0.1% w/w is added to the product to inhibit the growth of microorganisms such as *Staphylococcus aureus*, *Staphylococcus epidermidis*, *Pseudomonas aeruginosa*, *Escherichia coli*, antibiotic resistant Methicillin Resistant *Staphylococcus aureus* (MRSA), and fungus *Candida albicans* within the product.

Indications

BIAKŌS™ Antimicrobial Skin & Wound Cleanser is intended for mechanical cleansing and removal of debris, dirt and foreign materials, including microorganisms from wounds such as stage I-IV pressure ulcers, diabetic foot ulcers, post-surgical wounds, first- and second-degree burns, grafted and donor sites.

Warnings

For external use only on skin and wounds. Do not use for infusion or injection. Do not ingest. Do not use in eyes. Do not contaminate spray nozzle.

Storage

Store with nozzle in closed position at room temperature up to 25°C (77°F). Avoid excessive heat. Do not refrigerate or freeze. Discard 56 days after opening.

Instructions For Use

Skin Cleansing

Spray intended area or wet through sterile gauze and apply to site. Air dry for 2 minutes or maintain as a wet dressing.

Wound Cleansing

Debride wound, if necessary. Spray BIAKŌS™ Antimicrobial Skin & Wound Cleanser onto entire wound bed, including the

wound edges. Avoid pooling. Alternative: Wet through sterile gauze pad with BIAKŌS™ Antimicrobial Skin & Wound Cleanser and apply to wound site. Cover wound site with a sterile gauze or other appropriate wound dressing. Repeat procedure once a day. Ensure that wound bed remains moist between dressing changes.

Ingredients

BIAKŌS™ Antimicrobial Skin & Wound Cleanser contains Purified Water, Poloxamer 407, Sodium Chloride, Ethylhexylglycerin, Hypromellose, Octane-1, 2-diol, Polyaminopropyl Biguanide (Preservative: PHMB: 0.1% w/w), Edetate Disodium [EDTA], Edetate Trisodium [EDTA].

Contraindications

BIAKŌS™ Antimicrobial Skin & Wound Cleanser should not be used if there is a history of allergy to any of the ingredients. Do not use in the presence of hyaline cartilage. Mixing BIAKŌS™ Antimicrobial Skin & Wound Cleanser with other wound cleansing soaps, lotions, ointments, oils, or enzymes may lower effectiveness. When such substances need to be removed from a wound, ensure the entire wound area is thoroughly rinsed with BIAKŌS™ Antimicrobial Skin & Wound Cleanser.

Adverse Reactions

In very rare cases there may be a mild burning sensation after application which will go away after a few minutes.

Caution

Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner.



Single use



Consult instructions for use



Do not use if package is damaged



Store at room temperature

Distributed by:

Wound Care Innovations, LLC,
1200 Summit Avenue, Suite 414
Fort Worth, TX 76102
817-529-2300

BIAKŌS is a trademark of
Wound Care Innovations.
© 2019. All rights reserved.
Patent on file. Rev. 200902
11033R0319

[Learn more at biakos.net](http://biakos.net)